SD BIO LINE Dengue Duo Dengue NS1+ Ab Combo

Test Procedure Testprocedure Testprosedyre Testprocedur

1

EN Now, open the package and look for the following:

1. Test device individually foil pouched with a desiccant
2. Assay diluent
3. Instructions for use
4. Capillary Pipette 10µl
5. Disposable dropper

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2

EN FIRST, read carefully the instruction on how to use the SD BIOLINE Dengue Duo test kit.

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3

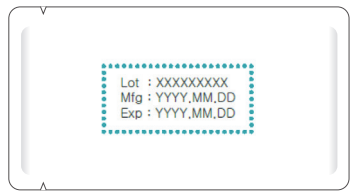
EN Next, look at the expiry date at the back of the foil pouch. Use another kit, if expiry date has passed.

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[For example / For eksempel / For eksempel / Exempel]

4

EN Open the foil pouch and look for the following.

1. Result window
2. Sample well
3. Assay diluent well

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
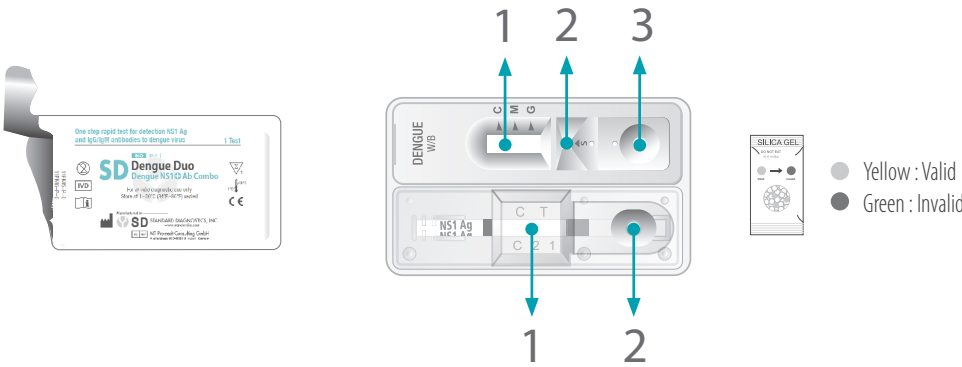
5

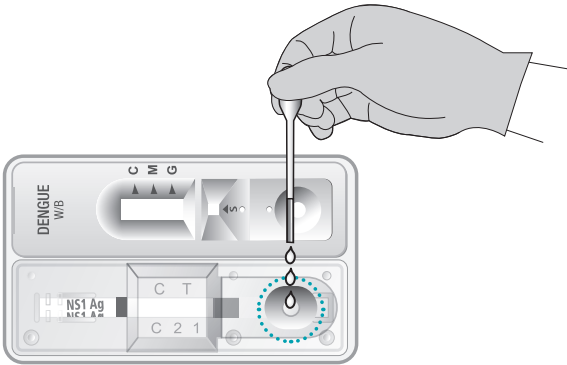
EN Using disposable dropper provided, add 3 drops (about 100µl) of serum, plasma or whole blood into the sample well.

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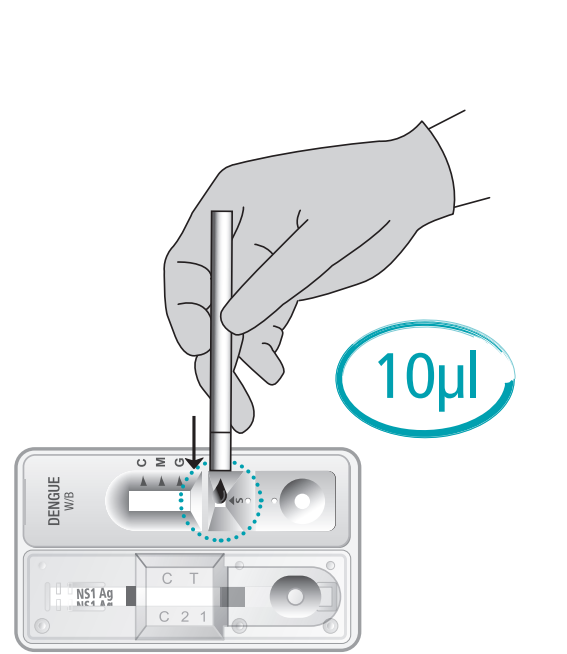
EN Using capillary pipette provided, add 10µl of serum, plasma or whole blood into the sample well marked "S".

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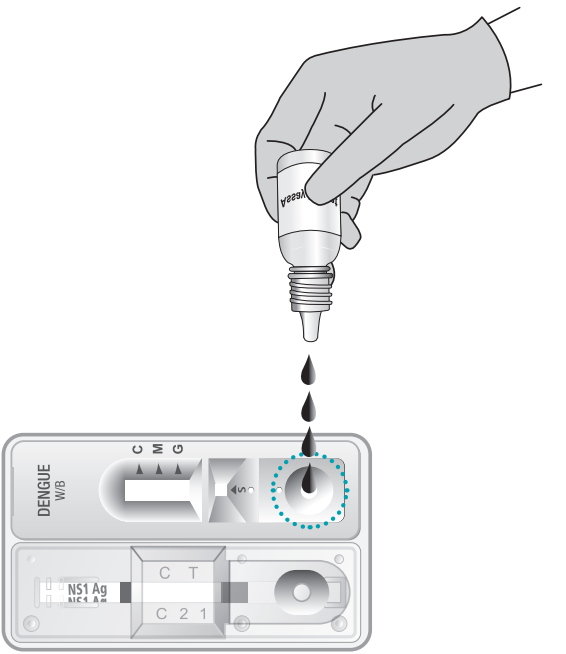EN Put 4 drops (about 90~120µl) of assay diluent into the round shaped assay diluent well.

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6

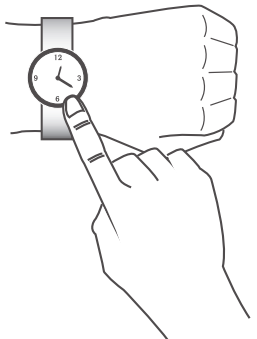
EN Interpret test results in 15~20 minutes. Do not read test results after 20 minutes. Reading too late can give false results.

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
Dengue NS1 Ag

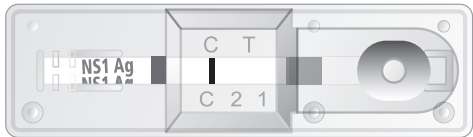Negative / Negativ / Negativ / Negativa


EN The presence of only one color line ("C") within the result window indicates a negative result.

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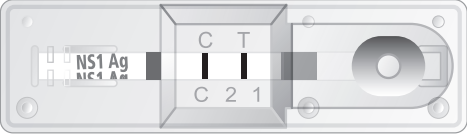Positive / Positiv / Positiv / Positiva


EN The presence of two color lines ("T" band and "C" line) within the result window in left-side, no matter which line appears first, indicates a positive result.

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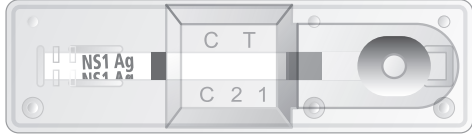Invalid / Ugyldig / Ugyldig / Ogiltigt

EN - No "C" line in result window.
- It is recommended that the specimen be re-tested.

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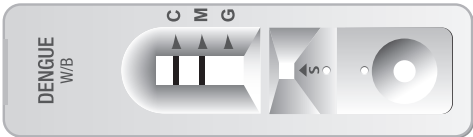
Dengue IgG / IgM

1. IgM positive / IgM positiv / IgM-positiv / Positivt IgM


EN (Primary dengue infection)
- Two lines "C" and "M" in result window.
- It is positive even if "M" line is weak.

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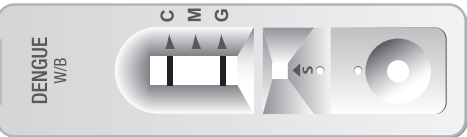2. IgG positive / IgG positiv / IgG-positiv / Positivt IgG


EN (Secondary or past dengue infection)
- Two lines "C" and "G" in result window.
- It is positive even if "G" line is weak.

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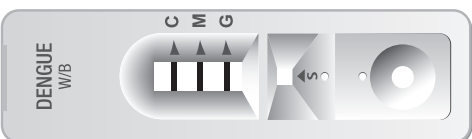EN (Late primary or early secondary dengue infection)
- Three lines "C", "M" and "G" in result window.

DA (Sen primær eller tidlig sekundær dengueinfektion)
- En "C"- , en "M"- , og en "G"-linje i i resultatvinduet.

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